Tedequim

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www.tedequim.com

Rev. B

NEUTRODENT®
2.0% Neutral Sodium Fluoride
Topical gel
Red berry flavored.

Fluoride at neutral pH is recommended for post-teeth whitening desensitization, it is ideal because it does not stain as it is colorless. It is also recommended for cavities without exposed pulp, for patients who do not tolerate acids, for those who have crowns, bridges and composite restorations, for those with exposed roots, for cases with gum inflammation, etc. Quick application sodium fluoride gels are acidic and cannot be used in the cases mentioned above.

Indications:

The neutral sodium fluoride is recommended for caries prevention and dental desensitization. Fluoride at neutral pH is recommended for post-teeth whitening desensitization, it is ideal because it does not stain as it is colorless. It is also recommended for cavities without exposed pulp, for patients who do not tolerate acids, for those with crowns, bridges and composite restorations, for those with exposed roots, for cases with gum inflammation, etc.

Instructions for use:

For topical applications, remove dental plaque and dry the surface with oil-water-free compressed air. Fill 1/3 of the bucket with gel and apply. Allow NEUTRODENT to remain in place for 4 minutes.

Remove the bucket and eliminate the surplus without washing. Warn the patient not to eat or drink within 30 minutes.

Content:

2.0% Sodium Fluoride.
pH = 6,8.

Side effect:

If after application the patient manifests an allergic reaction to the gingiva, (the product must be placed on enamel or dentin) the gel can be removed with rinsing and plenty of water.

Handling care:

- 1- Do not place excess gel in the bucket. Avoid overflows.
- 2- Keep in a cool place.
- 3- Keep out of the reach of children.

Recomentations:

Inform the patient that they should not eat food for 30 minutes, they can drink liquids other than citrus juices or soft drinks.

Applications for preventive purposes should be repeated every six months. In case of greater cariogenic activity, it is advisable to carry out the topicalization every three months.

In cases of teeth whitening, if the patient has sensitivity, it is advisable to eliminate the sensitivity before teeth whitening treatment. For patients who do not have sensitivity, it is recommended to apply it after the whitening treatment to prevent sensitivity.

Warmings:

If after application the patient manifests an allergic reaction to the gingiva (the product must be placed on enamel or dentin), the gel can be removed with rinses.

Do not apply in patients with:

- Ulcerative gingivitis.
- Stomatitis.

For its elimination, the bucket is removed and the patient has to eliminate the excess in the spit without rinsing, which is why it is not risky.

Storage:

Keep between 15°C and 30 °C. Do not store near heat sources

Autorized by ANMAT PM 1277-21

Technical director: Miryam della Vedova - Pharmacist - Licence 2107

Exclusive sale to professionals and sanitary facilities

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