**Tedequim**

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Rev. B

## TEDEQUIM® CAVITY VARNISH

Varnishes are fluid substances that are placed on the tooth to form a layer or film, in order to protect the cavity and pulp against the irritating action of permanent filling materials. They form a film resistant to acid attack and prevent leaks.

TEDEQUIM Cavity Varnish also provides antibacterial action and fortifies dentin due to the presence of calcium. The preparation of the cavity with the rotating instruments leaves bacteria residues and decalcification occurs, so that in addition to forming an insulating film that is the main function of the varnishes, it has a bactericidal agent that eliminates the rest of the bacteria and calcium that remineralizes secondary dentin.

It is used: a) in enamel or dentin cavities which are going to come into contact with an amalgam filling, thus avoiding the penetration of metal ions through the dentin canaliculi, which produce a grayish coloration in the tooth around the filling; b) when fillings are to be made with materials containing acids, such as zinc phosphate cements, to prevent their penetration through the dentin canaliculi, thus avoiding the death of the pulp; c) in deep cavities, close to the pulp; d) should be placed before the cohesive gold, to reduce the action of galvanic currents.

It is recommended to use the varnishes after placing calcium hydroxide cements, zinc and eugenol cements, polycarboxylate cements to allow the medicinal action of these materials on dentin and dental pulp.

### Instructions for use:

1. Clean the walls of the cavity and surroundings with CAVITARY CLEANER
2. Air dry.
3. With sterilized tweezers, impregnate a cotton swab, brush, swab, etc. with Varnish for Cavities.
4. Apply on the surface to be treated.
5. Air dry for 10 seconds.

### Precautions - Warnings and special cares:

- Wear gloves.
- Isolate the tooth to be treated with cotton rolls, use an ejector, etc. to keep the cavity to be treated dry.
- In the case of the use of varnishes in general for cemented post placement procedures, these reduce the adhesiveness of the posts by a percentage close to 40%.
- In cases of acid demineralization, the varnish prevents contact of the filling material with the enamel and dentin walls, reducing the adhesion of the filling material to the tooth.

- They should not be used under acrylic resins or composite resins because they can react with the resin, they can soften it and the monomer dissolves the varnish.
- Cover immediately after use.
- Keep out of the reach of children.
- Keep away from heat sources.

**Composition:**

Quaternary ammonium, calcium salt and natural resin in alcoholic solution.

**Presentation:**

Glass dropper container containing 10 ml.

**Storage:**

Store the product well covered in a cool place, at temperatures between 4 and 30 °C.

Autorized by ANMAT PM 1277-2  
Technical director: Miryam della Vedova - Pharmacist - Licence 2107  
Manufactured by TEDEQUIM SRL

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