Tedequim

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Rev. B

TEDEQUIM® CAVITY CLEANER

Dental surgical procedures leave a thin layer of waste on the surfaces generated by the rotating instruments, which together with saliva, blood, bacteria, collagen residues, etc. constitute the so-called "SMEAR LAYER".

CAVITY CLEANER is a solution to clean prepared surfaces before obturation. It is buffered at pH 7.4 and contains a very effective combination of substances to eliminate organic and inorganic residues. It is bactericidal and does not widen dentinal tubules.

CAVITY CLEANER with fluoride (Orange label) also has the advantage of remineralizing the surface of enamel or dentine through the penetration of fluoride ion. Decreases the possibility of decay and post-operative sensitivity.

Indications:

- On enamel or dentin surfaces and in deep cavities with pulpal problems, before filling.
- Before the temporary or permanent cementing of crowns and bridges.
- In endodontic treatments as irrigation solution.
- The professional will decide when to apply the product with sodium fluoride.

Use Instructions:

- Rinse and air dry the cavity or surface to be treated.
- Impregnate a swab or brush with CAVITY CLEANER.
- Rub the surface and let the product act for one minute.
- Remove excess and dry thoroughly before further treatment.

Cautions and Warnings:

1. Use disposable gloves.
2. Cover immediately.
3. Keep in fresh area.
4. If vitreous ionomers are used for the filling process, it is convenient to use fluoride-free CAVITY CLEANER (Blue label).
5. Keep away from children.

Composition:

Contains bactericidal agents, detergents, chelants, disinfectants and phosphate buffer to regulate at pH 7.4.

Orange label also contains 1% sodium fluoride.

Presentation:

Without fluoride: Blue label - 60 ml dropper bottle.

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With fluoride: Orange label - 60 ml dropper bottle.

Storage:

Do not store near heat sources or at temperatures above 30 °C

Authorized by ANMAT PM 1277-19

Technical director: Miryam della Vedova - Pharmacist - Licence 2107

Exclusive sale to professionals and sanitary facilities

Manufactured by TEDEQUIM SRL

Bv. De los Polacos 6136- Los Boulevares

X5147GGP Córdoba - Argentina

Tel: 54-3543-448260

www.tedequim.com.ar

Rev. B