Tedequim

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NaFRESIN®

Varnish with Sodium Fluoride

Fluorinated varnishes allow contact between fluoride ions and the surface of the enamel or dentine for longer than those achieved by other topical applications such as gels.

Clinical studies reveal a high concentration of fluorine in the form of fluoroapatite in the outer layer of the enamel after a varnish application.

The presence of fluoride makes the enamel or dentin more resistant to acid solution and thus less susceptible to caries. This product is of great importance in the prophylaxis of dental caries and in the treatment of tooth sensitivity.

NaFRESIN® is indicated in temporary teeth, permanent ones prone to decay and dentine hypersensitivity.

The varnish is applied in the interdental spaces of both sides, in sensitive area, in pits and fissures, in the edges of crowns or obturations, in fixed orthodontic appliances and gingival margins of fixed prostheses.

Instruction of use:

Remove bacterial plaque completely, rinse and dry teeth before application.

Remove the cap from the syringe and insert the applicator needle.

Place the product on all the elements without touching the gum. In the case of sensitive teeth, encircle the area. The patient can then close the mouth and the saliva ends up drying and adhering the varnish to the enamel or dentine.

Let it act for at least two hours. It is convenient to leave the varnish film on the tooth as long as possible to allow a better penetration of the fluoride ions.

The patient can then brush the teeth to remove it.

Care in handling:

1. Cover immediately after use.
2. Store in a cool place.
3. Keep out of the reach of children.
4. Discard the applicator needle.

Recommendations:

- Use gloves.
- Advice the patient not to eat or drink (citrus juices or sodas) while the varnish is acting.
- Recommended dose:
 - In temporary teeth: 0.3 ml
 - In permanent teeth: 0.8 ml
- Applications for preventive purposes must be repeated every six months. In case of greater cariogenic activity it is convenient to carry out the topicalization every three months.
- In sensitive necks the application can be repeated two or three times, depending on the severity of the case, with intervals of 3 or 4 days or until the patient shows absence of sensitivity.

Warnings:

If after the application the patient manifests an allergic reaction on the gum, (the product should be placed on enamel or dentin) the varnish film can be easily removed by brushing and then rinsing.

The color does not affect the product.

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The contents of the syringe reach for several applications so it is convenient to cover it well to prevent it from drying out.

Use an applicator for each patient.

Content of the syringe can be used for several applications so it is convenient to close it well to prevent it from drying out.

Contraindications:

- Allergy to rosin.
- Ulcerative gingivitis.
- Stomatitis.
- Bronchial asthma.

Composition:

Suspension of sodium fluoride in alcoholic solution of a natural resin. 50 mg of NaF / ml (5%).

Presentation:

Syringe containing 3 ml and disposable plastic applicators.

Storage:

Do not store near heat sources or at temperatures above 30°C.

Authorized by ANMAT PM 1277-20

Technical Director: Miryam Della Vedova - Pharmacist - License 2107

Exclusive sale to professionals and sanitary facilities

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